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## Operation Manual

## Content

Introduction .....	20
Power Supply .....	21
Description of Parts .....	22
Operation .....	23
Important Notes .....	25
Specifications.....	26
Battery Charge.....	27
Alert Signals .....	28
Precautions.....	29
Warranty .....	31


## Introduction

Thank you for choosing the alcohol tester  
**Ritmix RAT-730.**



- **Ritmix RAT-730** Personal Breathalyzer, is a hand held Professional Alcohol Analyzer. Using the electrochemical reaction for platinum vs ethanol molecules, Ritmix RAT-730 Personal Breathalyzer can give stable and reliable measuring technology.
- Wash your mouth with water before using. Abstain from smoking or drinking alcohol for at least 20 minutes.

## Power Supply

- Slide back the battery cover on the back of the device.
- Install the rechargeable battery into the battery compartment. All segments turn on and then go to Alcohol mode (DAC).
- When battery energy is too low to do the alcohol test, battery indicator flashes. Recharge the battery immediately by plug in the provided charging cable.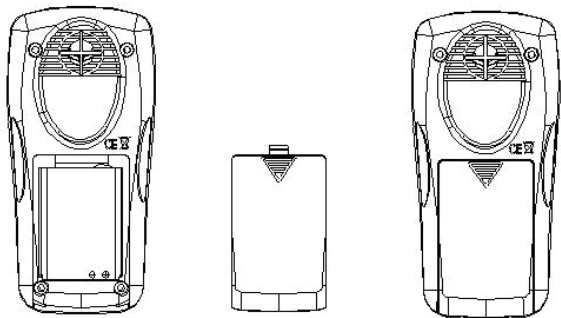

## Description of Parts

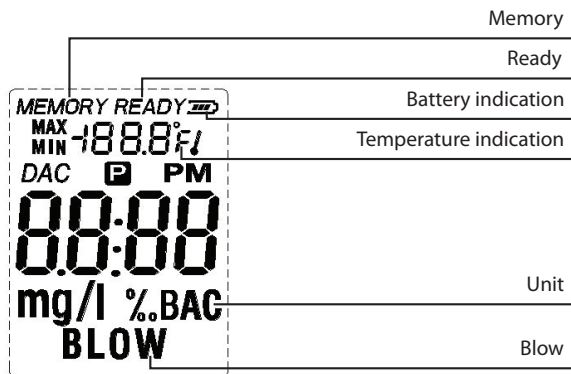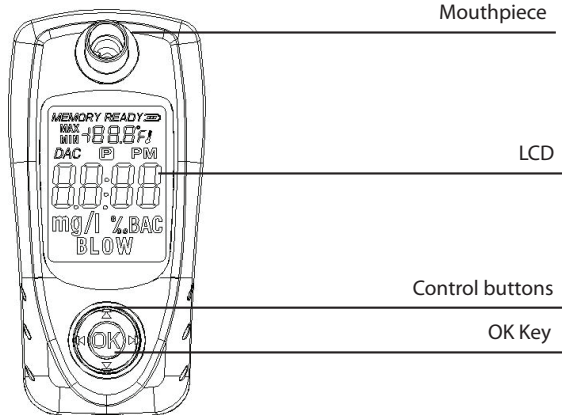
## Operation

- Press ◀ key for one second can turn on analyzer and press ◀ key for two seconds will turn off analyzer.
- Press the **OK** Key for approximately 2 seconds to activate sensor-warm-up process. **"PH"** flashes for few seconds and then **"READY"** & **"BLOW"** flash alternatively for about 60 seconds. User is required to perform the alcohol test within this period.
- When ready, user should blow continuously through the blow tube for approximately 5 seconds. Auto detect your breath sample (or press OK key once) and stop blowing after you hear a 'beep' tone. **"DAC"** flashes when analyzing the breath sample. The final result will be shown together with three blinking **"DAC"**.
- Your test result will be displayed on the LCD within 10 seconds.
- Hold ▼ key and **OK** key while install the battery into the battery compartment to change alcohol unit (%o BAC, mg/L, % BAC).
- Memory readings: press ▲ key to recall the last 25 memories. If memories full, **"MEMORY"** flashes. Press

and hold ▲ key for 1 sec and clear all memories. The **"MEMORY"** and **"Clr"** will flash. **"----"** shown on display means all memories are being erased.

- In DAC mode, press ▲ or ▼ key to scroll forward or backward the 25 memory readings. Hold ▲ key for 1 sec to clear the memory readings in DAC mode.

- The analyzer will turn off automatically after 2 minutes.

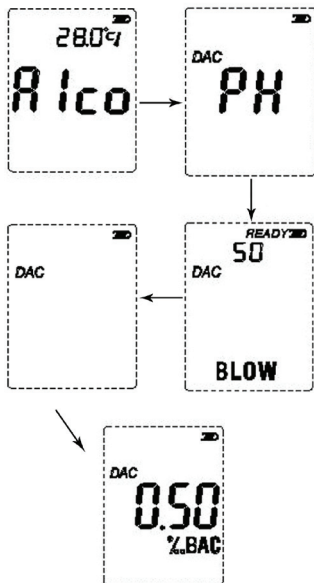## Important Notes

- When unused for a long time or at a low temperature, the machine will take longer to warm up time.
- After 5 consecutive tests, allow the device to rest for 2 minutes.
- Please store in a dry environment at room temperature.
- Avoid extreme heat or long term sun exposure..
- When battery indicator flashes, charge it.
- If the difference between two successive readings is  $>0.01\%$ ( $0.1\%$ ), user should test again.
- During the breath sampling, do NOT block the outlet.

## Specification

- Fuel cell Sensor
- Display Range: 0.000 ‰-4.000‰ (0.000%-0.400% BAC or 2.000 mg/l BRAC)
- Power Input: 3.7V Li-Battery
- Display: Digital LCD display with bi-color Backlighting.
- Auto Power Off: 2 minutes.
- Breath Sampling: about 5 seconds continuous breath sample @ 20 L/min flow rate.
- Dimension: 130\*29\*60 mm.
- Weight: 130 g (without battery).
- Operating temperature: -5 C<sup>0</sup> to +45 C<sup>0</sup>
- Agency Approval: CE
- Accessory: 2 mouth pieces, hand stripe, USB cable, car charger.

## Battery Charge

**Step 1.** Please connect the USB cable to the USB socket of tester, press and hold the ◀ key for 1 second to turn on the machine and activate battery charging.

**Step 2.** When charging, the  will flash. It will take 5-6 hours to charge full. After charge full, the  will stop flashing.

## Alert Signals

1. A measurement of less than 0.05‰BAC (0.023mg/l or 0.005% BAC) will show 0.000 ‰ BAC (0.000 mg/l or 0.000 % BAC) if the concentration detected is less than 0.05‰ BAC.

2. A measurement between 0.05‰ (0.023mg/l or 0.005% BAC) & 4.00‰BAC (1.904mg/l or 0.400% BAC) will show digital readings. The analyzer beeps. Red backlight flashes alternatively. The LCD will keep the RED backlight.

3. A measurement above 4.00‰BAC (1.904mg/l or 0.400% BAC), "HI" will show on the display. Red backlight flashes alternatively. The LCD will keep the RED backlight.

**YOU ARE PROHIBITED TO DRIVE IF YOU EXCEEDED THE ALLOWED FIGURES.**

## Precautions

Your blood alcohol content (BAC) may change within minutes

- Do not use this device to determine if you can drive.
- Do not share with friends. This unit is designed for personal use for accuracy and hygienic considerations.
- After drinking, it is recommended to last after 20 minutes, it is because 20 minutes needed approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate.
- For accurate result, do not repeat next testing within 3 minutes after last time.
- Do not smoke and eat for 20 minutes prior testing.
- Do not use chemicals to clean the tester directly. For cleaning the mouth piece please use the dry material or paper napkin.
- Do not blowing liquid into the breath pipe.
- Do not use any excitant substances near the tester, such as paint, insecticides, alcohol.
- Do not place the tester in the closed environment with contaminated air.
- Each individual's body responds differently to alcohol. Drink one glass of wine somebody can feel slightly drunken, somebody can feel heavily drunken.
- Diabetes or low-calorie diet or other factors are

leading to the rising of the concentration of ketones in exhalation and can cause false positive results.

- Any of Alcohol testers doesn't give the 100% of guarantee of the accurate results. The most accurate results give blood measuring test. That's why if the road patrol has stopped you, and alcohol tester showed the presence of alcohol in blood, require the blood test.
- Reports of World's Health Organization shows that if the alcohol concentration reaches the 0.08 g/ 100 ml, than the risk of road accidents increases in 2.5 times to compare if there is no alcohol in blood. If the alcohol concentration in blood reaches 0.1 g/100 ml, risk of road accidents increases in 4.7 times, than there is no alcohol in blood. Even if the slight quantity of alcohol is drunken it increases risk of road accidents in 2 times.
- The unit is designed to be used as an aid to promote reasonable driving. It should not be used in anticipation of the use or operation of vehicles, machinery, or equipment. The unit makes no references whatsoever to blood alcohol content or other such measures of intoxication promulgated by government, medical or other agencies or bodies. The result of this alcohol gives BAC for reference only; manufacturer does not to take any legal responsibility.
- Test results of alcohol tester are not probative.
- Don't drive in a drunk condition of any degree of intoxication.
- You cannot use the test results in Court.

## Warranty

The company-producer guarantees the normal work of device during the all warranty period if the device will be used according to the user manual.

Warranty period for this device is 6 months since the day of the selling through the commercial network. Time of service for this device is not less than 1 year.

The company-producer doesn't accept reclamations in case of the mechanical damage of device, in case of dismantle, wrong usage, or using not in a proper way. This device will serve you for many years if careful handling.

**Company «Ritmix»** has the right to change the Specification without notification.

[www.ritmixrussia.ru](http://www.ritmixrussia.ru)